



### Tools Required: Scissors

**1.** Locate the end of the strap where the ISKM legs cross. Using a pair of scissors, cut through the loop and remove the entire strap from the unit (Fig. 1) by un-threading it from the strap holder on the mesh.

NOTE: The non-removable strap ring (Fig. 2) must stay in place.

**2.** Insert end of replacement strap through strap ring with buttons facing outward (Fig. 3). Snap together and test that all snaps are engaged by pulling on the strap at the buckle (Fig. 4).

**3.** Repeat on other side.

**4.** Feed buckle end of straps through the strap loops on the mesh.

**5.** Feed buckle end of straps through the strap loops on the mesh.

